


Indications For Use:

RMS HlgH-Flo Subcutaneous Safety Needle Sets™ are intended for the delivery of medication to the subcutaneous tissue.

How to use HlgH-Flo needle sets:

1 Remove Cap

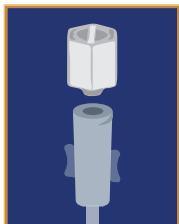


Remove the end cap from the female luer and connect the luer to the source of infusion.

2 Remove Closure



Remove the wings closure band to separate each wing.

3 Bend Back Wings

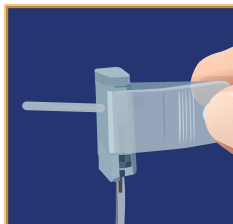


Bring the wings together and hold them between two fingers so that the needle guard is exposed.

4 Insert Needle




Remove the needle guard and insert needle into the subcutaneous tissue; cover with adhesive dressing.

5 Remove Needle

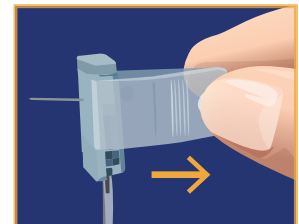

When infusion is complete, remove adhesive and pull needle out by bringing the wings together. Snap closed to enclose needle and discard.

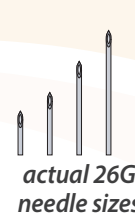

Safety Sets

- Safety closure protects against accidental needle stick injuries
- Emergency slide clamp included with every set
- Genuine 3M Tegaderm™ dressing to keep needles in place during infusion
- Made in the USA with highest level of quality control for defect-free product

Enhanced Design

- Backcut needle tip
 - *Minimizes tissue damage, scarring and local site reactions*
 - *Promotes faster healing*
- Custom luer provides even flow to every site
- Flexible flying hinge butterfly wings fit to patients body and keep needle in place without tenting

Versatile

- Needle lengths available in 4, 6, 9, 12, and 14 mm
- 24 and 26 gauge options
- Y-Connector combines sets up to 8 sites or used to infuse with 2 different needle lengths
- Sets available in single, double, triple, quad, penta and hexa configurations

Ordering Information

26 Gauge Needle Sets

1-Needle Sets box of 20

| Length | Item # |
|--------|----------|
| 4 mm | RMS12604 |
| 6 mm | RMS12606 |
| 9 mm | RMS12609 |
| 12 mm | RMS12612 |
| 14 mm | RMS12614 |



2-Needle Sets box of 10

| Length | Item # |
|--------|----------|
| 4 mm | RMS22604 |
| 6 mm | RMS22606 |
| 9 mm | RMS22609 |
| 12 mm | RMS22612 |
| 14 mm | RMS22614 |



3-Needle Sets box of 10

| Length | Item # |
|--------|----------|
| 4 mm | RMS32604 |
| 6 mm | RMS32606 |
| 9 mm | RMS32609 |
| 12 mm | RMS32612 |
| 14 mm | RMS32614 |



24 Gauge Needle Sets

1-Needle Sets box of 20

| Length | Item # |
|--------|----------|
| 6 mm | RMS12406 |
| 9 mm | RMS12409 |
| 12 mm | RMS12412 |
| 14 mm | RMS12414 |



2-Needle Sets box of 10

| Length | Item # |
|--------|----------|
| 6 mm | RMS22406 |
| 9 mm | RMS22409 |
| 12 mm | RMS22412 |
| 14 mm | RMS22414 |



4-Needle Sets box of 10

| Length | Item # |
|--------|----------|
| 4 mm | RMS42604 |
| 6 mm | RMS42606 |
| 9 mm | RMS42609 |
| 12 mm | RMS42612 |
| 14 mm | RMS42614 |



5-Needle Sets box of 10

| Length | Item # |
|--------|----------|
| 4 mm | RMS52604 |
| 6 mm | RMS52606 |
| 9 mm | RMS52609 |
| 12 mm | RMS52612 |
| 14 mm | RMS52614 |



6-Needle Sets box of 10

| Length | Item # |
|--------|----------|
| 4 mm | RMS62604 |
| 6 mm | RMS62606 |
| 9 mm | RMS62609 |
| 12 mm | RMS62612 |
| 14 mm | RMS62614 |



3-Needle Sets box of 10

| Length | Item # |
|--------|----------|
| 6 mm | RMS32406 |
| 9 mm | RMS32409 |
| 12 mm | RMS32412 |
| 14 mm | RMS32414 |



4-Needle Sets box of 10

| Length | Item # |
|--------|----------|
| 6 mm | RMS42406 |
| 9 mm | RMS42409 |
| 12 mm | RMS42412 |



Low Residual Volume Y-Connector - box of 50


Item # LRVY Combine sets up to 8 sites or infuse with 2 different needle lengths.

Residual Volumes

| 26 Gauge | 1 needle | 2 needle | 3 needle | 4 needle | 5 needle | 6 needle |
|-----------|----------|----------|----------|----------|----------|----------|
| Res. Vol. | .1 ml | .2 ml | .3 ml | .4 ml | .5 ml | .6 ml |

| 24 Gauge | 1 needle | 2 needle | 3 needle | 4 needle |
|-----------|----------|----------|----------|----------|
| Res. Vol. | .4 ml | .7 ml | 1.1 ml | 1.4 ml |

Other Products:

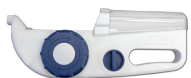

FREEDOM60® Syringe Infusion System
for larger infusion doses, 60 ml syringe



FreedomEdge® Syringe Infusion System
for smaller infusion doses, 20 ml and 30 ml syringes



RMS Precision Flow Rate Tubing™
for flow rates from 0.5 to 2400 ml/hr



24" Extension Tubing Set (FEXT)
for those who need a longer length